



REF 503750 - Bulk Syringe

REF 502450 - Restorative Kit

Description:

CHX is a 2% Chlorhexidine Gluconate Solution in water. CHX is a root canal cleanser for use in endodontic procedures. After endodontic instrumentation, these products should be used to cleanse the canal space before placement of the endodontic filling.

Indications:

CHX is used as a final endodontic rinse after instrumentation to irrigate and cleanse the root canal system for long lasting cleansing.

Instructions for Use:

1. For 1.2ml syringe:
 - a. Remove luer cap.
 - b. Securely attach tip.
 - c. Verify flow prior to intraoral use.
2. For 30mL Prefilled Syringe:
 - a. Remove luer cap from 30mL prefilled syringe.
 - b. Attach one end of the docking port to the 30mL prefilled syringe and attach the other end of the docking port to a 1.2mL syringe.
 - c. Slowly depress 30mL prefilled syringe plunger while guiding 1.2mL syringe plunger to desired fill.
 - d. Separate syringes from the docking port and re-cap 30mL prefilled syringe.
 - e. Securely attach tip to 1.2mL syringe.
 - f. Verify flow prior to intraoral use.
 - g. Clean and disinfect 30mL prefilled syringe between patients.
3. Once canal has been cleaned and shaped to appropriate size, rinse with CHX.
4. Air dry.
5. Obturate with materials of choice.
6. Dispose of tip and unit dose syringe.



WARNING: May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Wear protective gloves/ eye glasses. Follow all government regulations for waste disposal. Prevent release to the environment if possible. Do not flush waste into sewer or waterways. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash off with plenty of soap and water.

Precautions:

1. Read SDS for material handling and safety information.
2. Proper treatment planning and establishment of working length is required prior to use of this product.
3. Following standard of care, isolate the tooth prior to use of this product to prevent unintended tissue contact or accidental ingestion.
4. Properly evacuate the canal during use of this product to prevent unintended tissue contact or accidental ingestion.
5. It is recommended to use irrigation tips with lateral venting (27ga or smaller) to ensure effective distribution of irrigation solution throughout the root canal and minimize the risk of apical extrusion.

Warnings:

1. Keep out of the reach of children.
2. Avoid contact with skin and eyes.
3. Over-instrumentation prior to use of this product can result in unintended tissue contact.
4. Under-instrumentation prior to use of this product can result in tip binding and subsequent apical extrusion. Do NOT bind the tip in the root canal.
5. If tip binding occurs during the procedure, or if resistance is felt during irrigation, discontinue irrigation until the tip binding issue is resolved.
6. NEVER apply excessive pressure when irrigating this product or any endodontic irrigant.
7. If apical extrusion of irrigant has occurred, seek medical attention immediately.
8. If ingested, do not induce vomiting. Drink a large portion of water and provide fresh air. Seek medical attention immediately.

** NOT TO BE USED AS AN ORAL RINSE **

- Tips are disposable. Do not reuse.
- To avoid clogging, always replace cap after use.
- Do not ingest material.
- Do not use on patients with known sensitivity to chlorhexidine.



Inter-Med, Inc.

2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com