



VISTA APEX

EDTA Solution

REF 317002 - Unit Dose

REF 503700 - Bulk Syringe

Indications:

EDTA solution (17%); necessary for the removal of the smear layer and dentin mud.

Instructions for Use:

1. Remove EDTA syringe.
2. Remove syringe cap and attach irrigation tip securely to syringe.
3. Irrigate as desired.
4. Dispose of irrigating tip and syringe.

Precautions:

1. Read SDS for material handling and safety information.
2. Proper treatment planning and establishment of working length is required prior to use of this product.
3. Following standard of care, isolate the tooth prior to use of this product to prevent unintended tissue contact or accidental ingestion.
4. Properly evacuate the canal during use of this product to prevent unintended tissue contact or accidental ingestion.
5. It is recommended to use irrigation tips with lateral venting (27ga or smaller) to ensure effective distribution of irrigation solution throughout the root canal and minimize the risk of apical extrusion.

Warnings:

1. Keep out of the reach of children.
2. Avoid contact with skin and eyes.
3. Over-instrumentation prior to use of this product can result in unintended tissue contact.
4. Under-instrumentation prior to use of this product can result in tip binding and subsequent apical extrusion. Do NOT bind the tip in the root canal.
5. If tip binding occurs during the procedure, or if resistance is felt during irrigation, discontinue irrigation until the tip binding issue is resolved.
6. NEVER apply excessive pressure when irrigating this product or any endodontic irrigant.
7. If apical extrusion of irrigant has occurred, seek medical attention immediately.
8. If ingested, do not induce vomiting. Drink a large portion of water and provide fresh air. Seek medical attention immediately.

**** NOT TO BE USED AS AN ORAL RINSE ****

- Always replace cap after use.
- Avoid contact with skin and eyes.

**** STORE AT ROOM TEMPERATURE ****

 **WARNING:** May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Follow all government regulations. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash off with plenty of soap and water.



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com



Rx ONLY

Made in USA

317002-I-V-ENG (8)